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
## 2. Sauger montieren

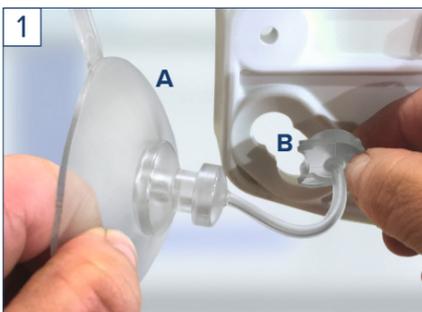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
1. Remove from the box


1. Lift the back rest      2. ...and remove.      3. Remove the seat unit.

2. Mount suction cups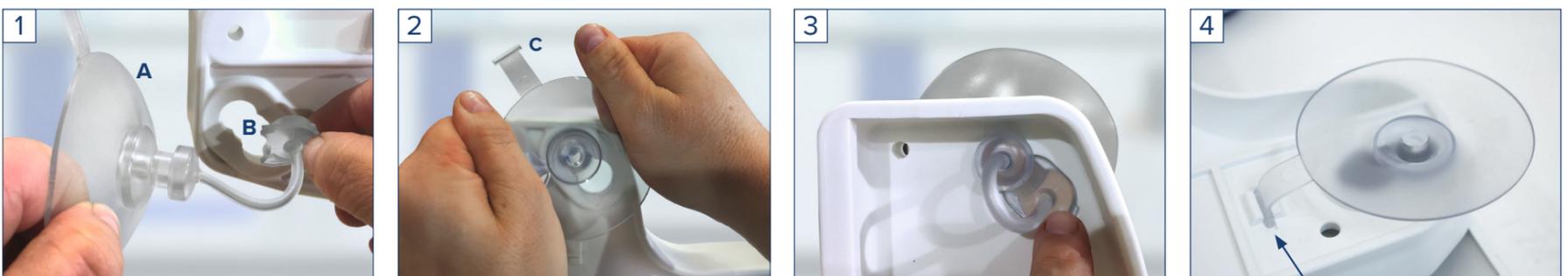


1. To insert the rear suction cups, begin by first inserting the movable plugs (B) into the underside of the base plate.  
2. Push the mushroom head (A) of the suction cup towards the small opening by gently rotating it into its final position. Make sure the vent tabs (C) are pointing downward.  
3. Push the plug from the upper side into the plate opening until it is flush.  
4. Insert the front suction cup in the same way. In this case, however, you must also insert the vent tabs into the base plate slots from the side. This will automatically vent the front suction cups during removal.

3. Insert backrest into seat unit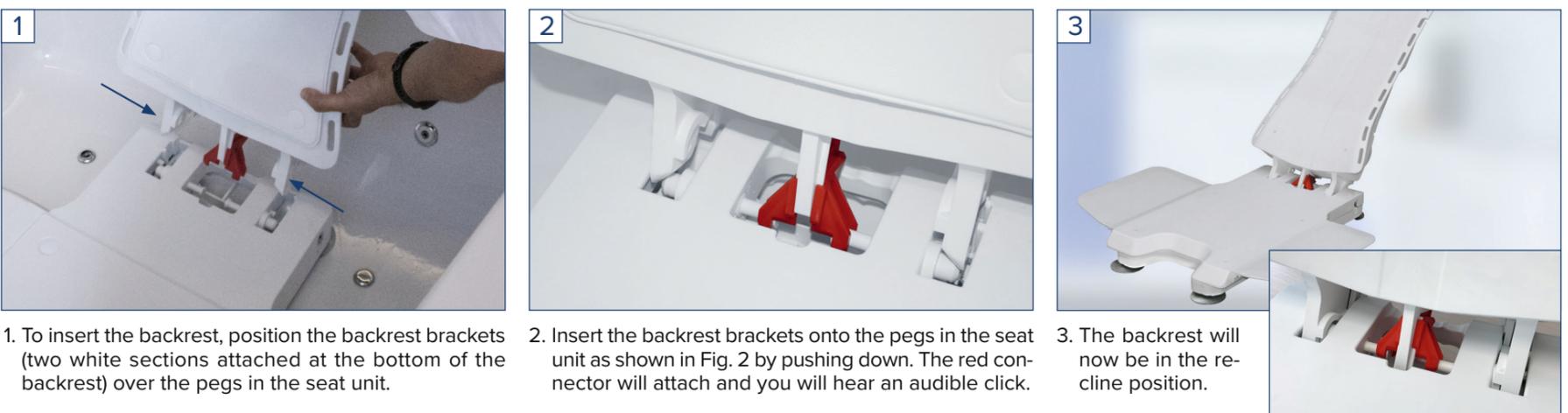

1. To insert the backrest, position the backrest brackets (two white sections attached at the bottom of the backrest) over the pegs in the seat unit.  
2. Insert the backrest brackets onto the pegs in the seat unit as shown in Fig. 2 by pushing down. The red connector will attach and you will hear an audible click.  
3. The backrest will now be in the recline position.

4. Charging the battery • Attaching the hand control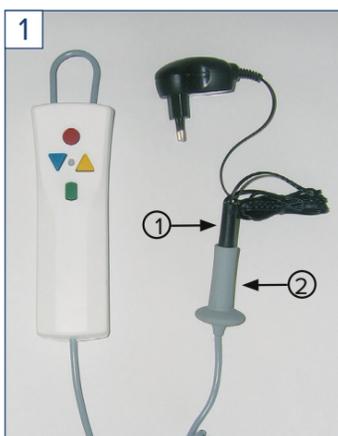


Charge the battery until the led indicator light turns green before using for the first time

1. Attach the charger to the operating hand control (simple jack plug operation) and then insert the charger into an electrical socket.  
The indicator light will be red.

Once the battery is fully charged (indicator light will be green), disconnect the charger from the electrical socket and then remove the operating hand control from the charger unit.

Do not place the operating hand control on a source of heat or in direct sunlight. Recommended storage at 15-20 degrees.

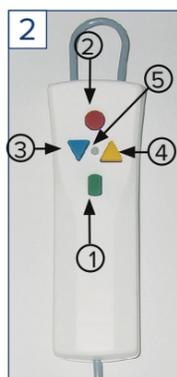2. Operating Hand Control Information:

- 1. Green: On button
- 2. Red: EMERGENCY OFF button
- 3. Blue: Down button
- 4. Yellow: Up button
- 5. Indicator light

Incorrect!

Correct!

3. Insert the jack plug (on the end of the operating hand control cable) into the connector on the top of the backrest. Ensure this is inserted fully (note Fig. 3) and until it noticeably clicks into place. If correctly inserted, the red ring is no longer visible

Please note : Do not clean using any of the products using highly concentrated chlorine-based cleaning agents