# Badelift bellavita

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
# Quick User guide - First installation

## 1. Remove from the carton and check stop lever


1. Raise the drive unit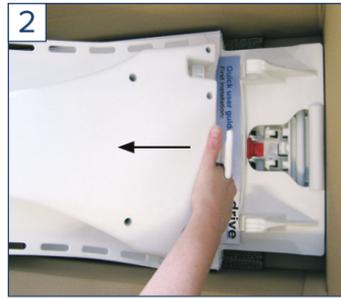

2. Move the backrest forward

3. Raise the seat unit

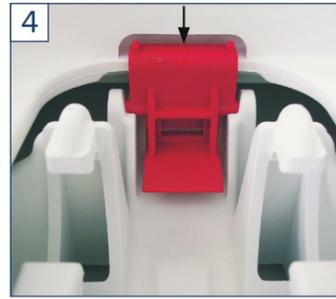
4. Please ensure that the red seat unit locking mechanism is engaged by pushing down on the seat.

## 2. Attaching the backrest into the seat unit


**1. Important:**


Please ensure that the actuator is fully retracted in the backrest.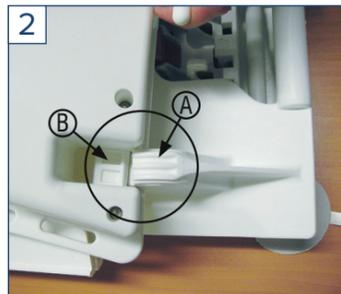

2. **A:** Bearing block – **B:** Guide pin  
Slide the backrest into the hinge located on the back of the seat unit.

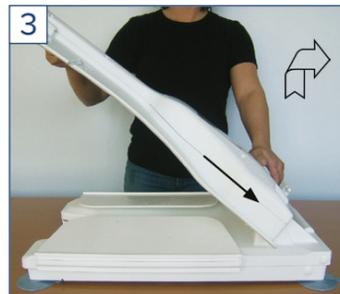
3. Lift the backrest to an 45° Angle until it clicks into the latch.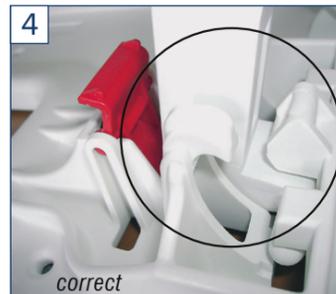


**4. Important:**

Please ensure that the actuator end is/sits properly in the base plate's ramp.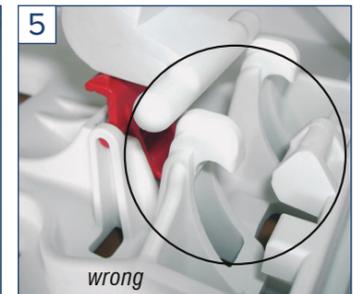

## 3. Charging the battery • Attaching the hand control

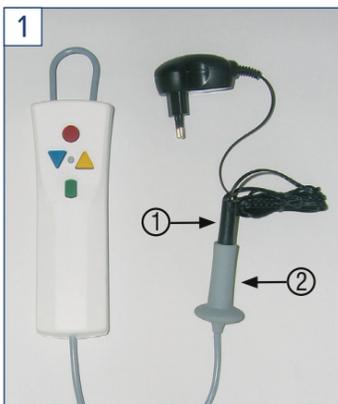
Recharge the battery after application.

1. Place the plug 1) of the charger into the socket of the cable 2) attached to the hand control. During the charging process, the red light will be on. The light changes to green when the battery is fully charged. After fully charging the battery, please disconnect the charger from the hand control.

Recommended value: Storage the hand control at room temperature.

Don't place the hand control at a heater. Please avoid direct sunlight.

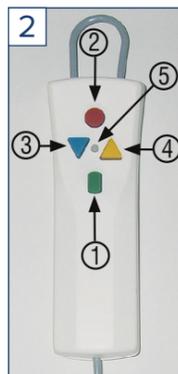
2. Hand control description:

1. Green: power On button
2. Red: emergency Stop
3. Blue: down button
4. Yellow: up button
5. Control light

3. Plug the hand control to the bathlift (at the back of the backrest). Make sure that the socket is properly seated on the jack and the red ring on the jack is now hidden.

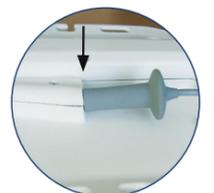Do not use high-dose chlorine cleaners for cleaning!

## 4. Placing or removing the bathlift into/from the bathtub

1.

- Insert the heads of the suction cups into the holes provided in the base plate and fix the end in the opening provided for this purpose.

2.

- The suction straps of the rear suction cups point outwards.
- The underside of the vacuum cups must be clean and dry, do not place slipping mats or other objects in the bathtub.
- Insert the seat unit into the empty bathtub so that the rear suction cups rest on the beginning of the sloping bathtub.

3.

- The suction straps of the front suction cups are fixed in the groove.
- Insert the backrest into the two bearing blocks of the seat unit (see point 2).
- **Folding the backrest backwards**  
**Backrest may not be used in the lowered** Do not place on the edge of the tub.
- Raise the lift to the top for use.

4.

- Ventilate the rear suction cups by pulling the straps at the same time.
- Pull up the underframe by the carrying handle and remove it from the tub.

## 5. Side flap protectors (pair) - Installation and Usage (Accessories)

1

The bathlift side protectors should be used on baths fitted with grab handles or similar which protrude inside the bath.

To install the protectors, snap the hooked end over the grab handle and allow the long end to slide down the edge of the bath.

Once installed, the side flaps on the bathlift will slide against the protector rather than the edge of the bath. This removes the risk of the bathlift side flaps getting caught underneath the grab handles.

For longer bath handles, one or more protectors may be installed (as shown on picture).

The product code is 400 570 and protectors are supplied in pairs.